



TIP SHEET

HOSPITAL INFORMATION SYSTEM (HIS)

HOW TO CLEAN THE WOW CART

1. Carts are meant to be 1-to-1 per shift and as long as the cart is not being shared between staff then wiping the cart at the start and end of shift is sufficient. Cleaning in addition to this should be performed as needed or when proper hand hygiene was not possible to be maintained.
2. Isolation rooms the card should remain outside the room
3. If room becomes a C-diff positive and that card had previously been rolled into that positive area, the cart must be cleaned with PCS 5000 to disinfect
4. For future reference here are the supported cleaning solutions that can be used on the cart

Recommended Cleaning Solutions

Quaternary Ammonium	Bleach-Based	Peroxide-Based	Alcohol-Based
<ul style="list-style-type: none"> • Metrex CaviWipes • Metrex CaviCide • PDI Sani-Cloth AF/Plus • PDI Super Sani-Cloth • EnVerros SaniMaster 4 • Virex 256 • DisCide • Safetec SaniZide Disposable Wipe Products • Envirocide 	<ul style="list-style-type: none"> • Bleach dilution (10%) • Clorax Dispatch 	<ul style="list-style-type: none"> • Hydrogen Peroxide • Virox 5 • Steriplex SD • Accel TB 	<ul style="list-style-type: none"> • Isopropyl Alcohol • Simple Green

Cleaning Agents and Chemicals to Avoid

- Non-Diluted Bleach or Chemicals
- Acetone
- Mineral Spirits
- Abrasive Cleaners
- Baking Soda
- Paint Thinners/Solvents
- Any Flammable or Toxic Chemicals